



## Instructions for **Resyncing** Hearing aids with iPhone

1. Select Settings
2. General
3. Accessibility
4. Hearing Devices
5. Click on Devices
6. Select forget this device
7. Clear out background apps
8. Power down cellphone
9. Open battery doors on aids
10. Turn phone back on
11. Select settings
12. General
13. Accessibility
14. Close battery doors on aids
15. Select hearing devices
  - » iPhone should find them and ask you to pair (2x)

**If still a problem, call Apple Accessibility and Assistive Technology number  
1-877-204-3930**